



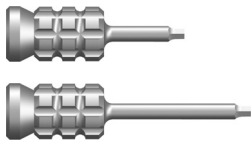

Abutment and bridge screws LASAK

Screwdrivers

The Lasak implant system features two main groups of fixing screws – the abutment screws and the bridge screws. For tightening of the abutment screws (as well as implant cover screws, healing cylinders and other components) use IMPLADENT screwdriver **hex 1.4**. To handle bridge screws which fit to Abutments for screw retained prostheses use IMPLADENT screwdriver **hex 1.0**.

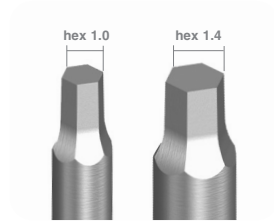

Hex hand screwdrivers intraoral

hex 1.4 /L11/L18, extra short	3224.3
hex 1.4 /L11/L25, short	2224.3
hex 1.4 /L21/L35, long	2524.3
for screw-retained prostheses hex 1.0 /L11/L25, short	5224.3
for screw-retained prostheses hex 1.0 /L21/L35, long	6224.3

To make sure the implant-abutment interface is really tight and to avoid loosening of a crown make sure to tight the abutment screw with help of Torque adapter (3024.3) attached to Ratchet (924.3). The easy-change reversible mechanism of the ratchet ensures a comfortable change of the working motion without the need to dismount the ratchet from the site. Tightening torque of abutment screws is generally 35 Ncm. Use new abutment screw for final placement of the abutment. For tightening of bridge screws and some other special components different tightening torques apply – please see below.




Ratchet	924.3
---------	-------



Torque adapter for ratchet	3024.3
----------------------------	--------

Screwdrivers for Ratchet

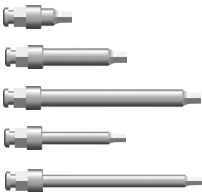

hex 1.4 /L4, extra short	24224.3
hex 1.4 /L11, short	4224.3
hex 1.4 /L21, long	4024.3
for screw-retained prostheses hex 1.0 /L11, short	44224.3
for screw-retained prostheses hex 1.0 /L21, long	24024.3

Tightening torque overview

- Abutment screws** / Screwdrivers **hex 1.4** (24224.3, 4224.3, 4024.3) / tightening torque 35 Ncm
- Temporary abutments PEEK** / Screwdrivers **hex 1.4** (24224.3, 4224.3, 4024.3) / tightening torque 20 Ncm
- Bridge screws** / Screwdrivers **hex 1.0** (44224.3, 24024.3) / tightening torque 15 Ncm
- Ball attachments** / Extension wrench (2024.3) / tightening torque 35 Ncm
- Locators** / Locator driver (09999) / tightening torque 35 Ncm
- Titanmagnetics abutment** / Dedicated drivers (H.00.04.X2, H.00.04.K2) / tightening torque 20 Ncm

There is set of prosthetic instruments containing ratchet and **hex 1.4** and **hex 1.0** screwdrivers available for loan – ask your sales representative.

Use new abutment or bridge screw for placement of the final prostheses.

