

# Washing recommendations for cleaning textiles

**Machine loading:** 70% nominal load

The specifications given in the recommendation should only be taken as a guide. Please, before putting them into practice, contact your supplier of dosing detergents.

	Time (min.)	Temperature	Water level	Products
Soaking	2	Cold	Medium	
Pre-washing	6	40 – 45° C	Medium	Detergent for pre-washing
Washing	10	60/70° C	Low	Detergent for the main wash
1st Rinse	3	30° C	High	
2nd Rinse	3	30° C	High	
3rd Rinse	3	Cold	High	
Neutralization	1	Cold	High	

	Time (min.)	Temperature	Others
Dry	7 – 8	85 – 90° C	Cool 2 minutes

Minimum amounts of detergents or cleaning products are sufficient. This fabric is easily washed. The rinse phase is especially important. Therefore, we recommend a larger dosage of detergent during the pre-wash and a smaller one during the washing process. This way, we facilitate its neutralization in the rinses.

To ensure the longevity of the properties of our technical fabrics, it is recommended to completely remove (neutralize) any detergents or alkalines. Detergent and alkaline remains lead to premature aging of the fibers.

- ! **Do not use fabric softeners, bleach, bactericidal products, fungicides, etc.**

